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ACE DA7100

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ACE DA7100

Author liability:

All information herein was created conscientiously and under assumption of correctness. The creator cannot be held liable though for the use of any kind whatsoever and any ensuing consequences. The creator is not liable for any costs or expenses for damage that are the result of any information within this document. Any information is subject to change without prior notice.

Short description:

Digital breathalyser with a sophisticated, electrochemical sensor for private and commercial use.

Scope of delivery:

- Digital breathalyser – ACE DA7100
- 2x AAA batteries (1.5 V)
- 6 mouthpieces
- Carrier bag
- Hand strap
- Operating manual

Introduction:

Thank you for purchasing our breathalyser ACE DA7100, which convinces by enormous reliability. The device can be used, for example, in schools and many other areas. It measures the estimated BAC (blood alcohol concentration) in human breath using an electrochemical, especially durable sensor and offers exact breath alcohol test results.

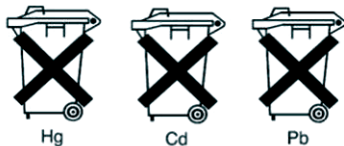
This way you receive a result as accurately as possible:

- a) Wait at least 15 minutes after your last sip of alcohol. Do not eat, drink or smoke during that time.
- b) Do not open the casing.
- c) Do not keep the ACE DA7100 in cold, hot, moist or badly ventilated places.
- d) Keep the ACE DA7100 out of the reach of children.
- e) Conduct three inlet tests after long storage before conducting the actual alcohol test.
- f) Do not let the ACE DA7100 fall.
- g) Change batteries after battery alarm. Diminished battery capacity leads to incorrect test results.
- h) Do not conduct tests in loud (sound waves disrupt the sensor), draughty or badly ventilated places.
- i) After a test result of 1.00 per mille or higher the sensor needs a long resting time. In this case wait at least three minutes before the next test.
- j) Do not conduct tests in very cold, very hot or unventilated, smoky ambience.
- k) Store the ACE DA7100 in a way to avoid penetration by dust or liquids.

Notes to the battery law (BattG):

The device for which this operating manual is contains two AAA alkaline batteries in delivery condition. Those batteries are not made for recharging. Trying to recharge can cause damage to property or persons. Also the use of rechargeable batteries is not possible with this device, because it can cause damage to the device. When the batteries are depleted, they must not be thrown in the household waste. Used batteries may contain harmful substances that can cause damage to the environment or health. Please return batteries / rechargeable batteries to the store or to the buyback centre. The return is free of charge and mandatory by law. Please throw only depleted batteries into the containers and tape over the poles of lithium batteries. For battery exchange only pull off the battery compartment cover and change the depleted batteries for new ones. Mind the polarity: The negatives poles of the batteries are in contact with the springs in the battery trough. Slide the battery compartment cover over the casing until it snaps in. All batteries and rechargeable batteries will be recycled. This way, precious resources like iron, zinc and nickel are gathered. Battery recycling is the easiest lesson in environmental.

The garbage bin means: Batteries and rechargeable batteries must not be thrown into the household waste. The signs below the garbage bins stand for:

- Hg: Battery contains mercury
- Cd: Battery contains cadmium
- Pb: Battery contains lead

Calibration:

To maintain the accuracy of the ACE DA7100, we recommend calibration twice a year. The calibration is not mandatory; the accuracy of ACE DA7100 will decrease steadily without it, though. Send the device with the note "Alkomat calibration service" to the company address stated on the invoice you received with the device. **We recommend sending it as a parcel, so that in cases of loss an inquiry or proof of delivery is possible.** Due to changes in the product, procedures stated can change. This can take place without prior notice or adaptation of the operating manual and is no cause for complaint, return or price reduction. The user is responsible to recognise deviations between the operating manual and the actual handling of the ACE DA7100 and he has to react accordingly.

Preparation for the test:

1. Install 2 AAA batteries in the battery compartment. Pull off the battery compartment cover at the back of the device carefully and insert 2 AAA (alkaline, 1.5 V). Mind the battery polarity in the process (+/-).
2. Put a mouthpiece on the mouthpiece connection, which is on the side of the device. Make sure it is attached properly. Use a different mouthpiece for every test person. Avoid blowing cigarette smoke into the tester!

Conducting the measurement :

- 1) Press the activation button. The overall number of all executed tests is displayed for a short while, and then a countdown starts.
- 2) As soon as the countdown reaches zero, a bleep is sounded twice, three lines (graphic) are displayed and the word "start" blinks. Take a deep breath and blow steadily for at least 5 seconds into the mouthpiece until you can hear two beeps again.
- 3) When the executed test was successful, moving LCD line appear at the rim of the display, as can be seen in the depiction.
- 4) After a few seconds (depending on the alcohol level) the estimated BAC (blood alcohol concentration) is displayed. The determined value blinks for several seconds. When you want to conduct another test, press the start button to reactivate the countdown.

Important: When you have eaten, drunken alcohol or smoked a cigarette, wait at least for 20 minutes before starting the measurement.

Display	Meaning	Action
	Full battery	No Action required.
	Depleted battery: Warning signal when battery is low. The device is operational, but to ensure correct results, you should insert new batteries.	Insert new batteries.
BAT	When "BAT" is displayed, the device is not operational. Change batteries immediately.	Insert new batteries immediately.
Flo	This is displayed when you have blown wrong or too weakly into the mouthpiece.	Press the start button again to repeat the test.
Out	When you have not blown into the mouthpiece within 30 seconds, this is displayed.	Press the start button to reactivate the countdown.
Out C	This is displayed when the device was started outside of the approved operating temperature (0 - 40° C).	Repeat the test as soon as the correct operating temperature is reached.

Technical data:

Statement of the BAC: 0.000 – 0.500% BAC (blood alcohol concentration)	Energy supply: 2 AAA alkaline batteries (1.5 V)
0.00 ~ 5.00 ‰ (fixed basic setting)	Battery lifespan: Approx. 1.000 Tests
0.00 ~ 2.50 mg/l	Dimensions of the device: 48 mm x 155 mm x 18 mm
Accuracy: +/- 0.05 ‰ for 1.0 ‰ (at 25° C)	Weight: 93 g (including batteries)
Warm-up time: Within 18 seconds	Calibration: Every 6 - 12 months or after 500 tests
Reaction time: Within 10 seconds with 0.05 ‰ BAC	Operating temperature: 0 - 40° C
Sensor: electrochemical sensor	

Warranty and liability:

Warranty:

The ACE Handels- und Entwicklungs GmbH will give a warranty of two years after purchase for material and processing errors for the product, to which this warranty and liability declaration is attached, in all versions and designations; a warranty of one year for the sensor. This warranty includes the electronics as well as the casing or the sensor without sampling components. Components that are inevitably destroyed upon effectiveness (such as fuses) are not part of this warranty. The warranty is valid from the date of delivery or from the date of first activation (always the shorter period), if this is not stated otherwise in the accompanying literature of the product by ACE Handels- und Entwicklungs GmbH.

Limitation of warranty and liability:

Product description: The ACE Handels- und Entwicklungs GmbH rejects any further expressly stated or implied warranty, including the marketability and suitability for a determined purpose. This warranty especially excludes calibration. After changing the sensor, calibration is absolutely necessary. When an exchange of the sensor or a calibration is necessary for a product, the user may receive an estimate of costs. Should the user refuse the estimate, the ACE Handels- und Entwicklungs GmbH will send the product back unpaired / uncalibrated and the product will be labelled accordingly. Should the product not fulfil the warranty, the claim of the buyer and the obligations of the ACE Handels- und Entwicklungs GmbH are limited exclusively and at the discretion of the ACE Handels- und Entwicklungs GmbH to an exchange or repair of the affected goods or to a reimbursement of the purchasing price for the faulty goods.

Application restriction: The ACE Handels- und Entwicklungs GmbH is on no account liable for additional damage, incidental damage or consequential damage such as, among others, lost profit and downtime that result from selling, producing or using products, which are part of this application restriction. This is independent of any claims, such as from contracts, negligence or other culpable actions.

The product, which is accompanied by this warranty and liability declaration, no matter in which version or designation, can especially not be used legally binding to determine capability to operate vehicles or to execute action that could lead to damage to oneself or others under the influence of alcohol.

The thorough check for damage of all products upon receipt, a calibration suitable for the given use by the buyer and the proper use, maintenance and service according to the instructions of the accompanying product literature by the ACE Handels- und Entwicklungs GmbH are mandatory prerequisites for this warranty. Maintenance and service work by non-qualified persons, as well as using non-approved consumables or spare parts lead to an expiry of this warranty. As for all technically challenging products, it is a fundamental prerequisite and mandatory for the warranty by ACE Handels- und Entwicklungs GmbH, that all users of these product are familiar with the properties and restrictions of the product as stated in the respective literature.

Advice restriction: The buyer confirms that he has checked the intended use and suitability of purchased goods for any purpose in sole responsibility. Between the two parties it is expressly agreed upon that all technical or other advice by ACE Handels- und Entwicklungs GmbH regarding use of the goods or services is free of charge and was made with the exclusive risk of the buyer. Therefore no obligation or liability can arise to the ACE Handels- und Entwicklungs GmbH from any recommendation or their result.

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